MCKESSON

Care and Handling of Surgical and Dental Instruments

Directions for Use / English

For professional use only.

INDICATIONS FOR USE

Surgical and dental instruments are designed to perform specific functions such as cutting, grasping, clamping, dissecting, probing, retracting, draining, aspirating, suturing, or ligating.

CONTRAINDICATION

Instruments should not be used for anything other than their intended use.

WARNING

If any device is/was used in a patient with, or suspected of having Creutzfeldt Jakob Disease (CJD), the device cannot be reused and must be destroyed after exposure as the instruments have not been validated to withstand the chemical and thermal exposure recommended to eradicate prions.

PRECAUTIONS

- McKesson instruments are supplied non-sterile, unless otherwise noted and must be cleaned, lubricated and sterilized prior to use according to hospital protocol and procedures outlined in this document.

DECONTAMINATION AND STERILIZATION PROCEDURES

Personnel should follow accepted guidelines as recommended in ANSI/AAMI ST79 - Comprehensive guide to steam sterilization and sterility assurance in health care facilities and ISO 17664. For Ethylene Oxide sterilization, follow the ANSI/AAMI ST41 Ethylene Oxide sterilization in health care facilities: safety and effectiveness, 4th edition.

CARE AND HANDLING

1. PRECLEANING

- For ultrasonic cleaning, moist and do not allow blood and/or bodily fluids to dry on the instruments.

3. CLEANING

- If appropriate, disassemble surgical instruments prior to cleaning and sterilization.

A. MANUAL CLEANING:

- To prevent formation of biofilm, cleaning should occur as soon as possible after instrumentation is used.

B. ULTRASONIC AND MECHANICAL CLEANING

- For ultrasonic cleaning, follow manufacturer's specifications for water level, concentration levels of cleaning agent and temperature.

4. RINSE AND DRY:

- Rinse all instruments thoroughly with distilled, pyrogen-free water to remove all traces of debris and cleaning agents.

5. LUBRICATION

- After thoroughly cleaning instruments, proper application of lubricants to all joints and movable mating surfaces will keep them moving freely and aid in protecting surface from mineral deposits.


6. STERILIZATION

- After following decontamination recommendations, reusable instruments are ready for sterilization.

Table with 4 columns: Sterilization Method, Temperature, Exposure Time, Dry Time. Includes rows for High Vacuum Steam and Wrapped.

Table with 4 columns: Sterilization Method, Temperature, Exposure Time, Dry Time. Includes rows for Unwrapped in containers, Gravity Steam, and Ethylene Oxide (EtO).

SYMBOLS USED ON LABELING

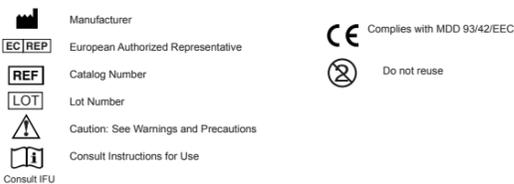Distributed by McKesson Medical-Surgical Inc. Richmond, VA 23233 PVN A0217 Rev. AA 03/2017 MCK-GENINSTDFU

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